

Diversey

Avert™ Sporicidal Disinfectant Cleaner



A ready-to-use one-step defense against Clostridium difficile (C. diff) spores on hard, non-porous surfaces such as those found in hospitals and other healthcare facilities.

Features

- Effective in 4 minutes against Clostridium difficile (C. diff) spores.
- 1 minute contact time effective against a broad spectrum of pathogens including bacteria, antibiotic resistant bacteria, viruses, and fungi.
- 12,500ppm chlorine exceeds CDC, SHEA, APIC and AORN recommendations.

Benefits

- Cleans, disinfects, and deodorizes in one labor saving step.
- Bleach-based formula means no quat-binding concerns.
- No rinsing required after use.
- Fresh, clean scent in a gentle, low-odor formula.

Applications

- Hard surface spray & wipe cleaning
- Safe use on hospital-grade surfaces including stainless steel, porcelain, glass, plastic and glazed ceramic.

Avert™ Sporicidal Disinfectant Cleaner



Use Directions

Special Instructions for Cleaning Prior to Disinfection against Clostridium difficile spores:

Personal Protection:

 Wear appropriate barrier protection such as gloves, gowns, masks or eye covering.

Cleaning Procedure:

Fecal matter/waste must be thoroughly cleaned from surfaces/
objects before disinfection by application with a clean cloth, mop,
and/or sponge saturated with the disinfectant product. Cleaning
is to include vigorous wiping and/or scrubbing, until all visible soil
is removed. Special attention is needed for high-touch surfaces.
Surfaces in patient rooms are to be cleaned in an appropriate
manner, such as from right to left or left to right, on horizontal
surfaces, and top to bottom, on vertical surfaces, to minimize
spreading of the spores. Restrooms are to be cleaned last. Do not
reuse soiled cloths.

Infectious Materials Disposal:

 Materials used in the cleaning process that may contain feces/ wastes are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

Procedure for Disinfection of C.difficile Spores:

- Apply product by coarse trigger sprayer to hard, nonporous surface.
- Spray 15-20 cm from surface, thoroughly wetting surface.
- Let treated surface remain wet for 4 minutes.
- Wipe surfaces and let air dry.

For Use as a One-Step Cleaner/Disinfectant, Deodorizer, and Fungicide:

- Pre-clean heavily soiled areas.
- Apply product by coarse trigger sprayer to hard, nonporous surfaces.
- Spray 6–8 inches from surface, thoroughly wetting surface.
- Let treated surface remain wet for 1 minute.
- Wipe surfaces and let air dry.

Product Specifications

Description	
Certifications	EPA 70627-72
Color/Form	Pale Yellow, Liquid
рН	12.5
Scent	Chlorine
Shelf Life	1 year

Available Items

Product Code	Description/Package Size	Dilution
100842725	12 x 32 oz./946 mL Spray Bottles	Ready-to-Use

Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available on-line at www.sealedair.com or by calling 888 352 2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, http://naextranet.diversey.com/dot/, for up to date shipping information.