



# Resound Multi Mic

## TIPS & TRICKS



*ReSound's Multi Mic functions identically to Cochlear's Mini Microphone 2+.*  
*These devices transmit audio from a variety of sources directly to personal devices (up to 82 feet).*  
*This product is compatible with Resound hearing aids and Cochlear sound processors.*

### PAIRING

*After the initial pairing, devices will stay connected to the Multi Mic.*

1. Turn the hearing aids/sound processors off.
2. Turn the Multi Mic on. The indicator light will flash **green** to confirm Multi Mic is powered on.
3. Press the pairing button on the Multi Mic. The indicator light will flash **yellow** to confirm pairing mode. The Multi Mic is in pairing mode for 20 seconds.
4. Turn the hearing aids/sound processors on. The child will hear a melody once devices are paired.
5. An adult should always confirm successful connection with a listening check.



### CHANGE FUNCTION

*Multi Mic functions including microphone, telecoil streaming, external audio streaming, and FM connectivity.*

1. Locate the Mode of Operation button on the back of the Multi Mic, below the pairing button.
  2. Press this button once to confirm the current function. An indicator light will blink next to the corresponding icon (microphone, telecoil, external audio or FM).
  3. To change function, press the button until the light indicates desired function.
- Note: Line-In and FM Modes can only be selected if an audio cable or FM receiver are connected.

View this [Step-by-Step Video](#) for additional support.




# Resound Multi Mic

## TIPS & TRICKS



### REMOTE MICROPHONE STREAMING

1. Turn on the Multi Mic and the hearing aid/sound processors.
2. Press and hold the button on the sound processor (3 seconds) to put the device into streaming mode.
3. The child will hear a melody, confirming active streaming. An adult should always confirm a successful connection with a listening check.
4. To mute/unmute the Multi Mic, press the button between volume up/down on the Multi Mic. An indicator light below the mute/unmute button will flash **yellow** while muted.

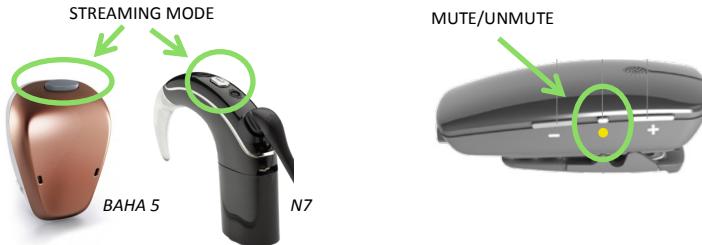

### CONNECTING TO ROGER

*Connecting only needs to be done once unless the Roger X receiver is connected to another transmitter.*

1. Turn on the Roger transmitter and the Multi Mic.
2. Plug the Roger X receiver into the bottom of the Multi Mic.
3. Press Connect on the transmitter. Hold the Roger X receiver within 1 inch of the transmitter.
4. Check the mode of operation indicator light to confirm the Multi Mic automatically switched to FM mode.  
Once connected, the device will remain connected for future use.
5. Ensure the sound processors are in streaming mode, and always confirm the connection with a listening check.



# Resound Multi Mic

## TIPS & TRICKS



### STREAMING EXTERNAL AUDIO

*Listen to audio from phones, computers, laptops, and tablets, etc. with the line-in cable provided in Multi Mic kit.*

1. Plug the mini jack cable into the bottom of the Multi Mic next to the FM connector.
2. Plug the other end into the audio out of the external device (i.e. laptop, phone).
3. Ensure the hearing aids/sound processors are switched into streaming mode.
4. Check the Mode of Operation indicator light to confirm Line-In is selected.



### CONNECTING TO TELECOIL

*Telecoil (T-Coil) function may be available at large venues such as theaters, churches, and auditoriums.*

*The telecoil feature allows users to stream the main speaker's voice to the sound processors.*

1. If an environment has telecoil capabilities, turn on the Multi Mic.
  2. Switch the sound processors into streaming mode.
  3. Press the Mode of Operation button until the indicator light confirms Telecoil mode.
- Note: Venues that have telecoil capabilities will often display a sign to notify individuals.

